



1. Attach only the “back” side of the SOS Collection Bag to the SOS (The side closest to the body) by slipping the holes in the Collection Bag over the clips on the SOS driver body.

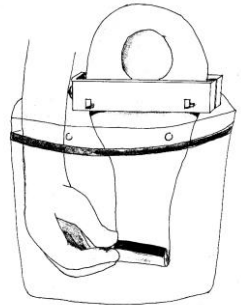

2. Open the SOS and place it as high as practical on the colostomy pouch and snap it closed. Reach in, thru the front of the SOS Collection Bag and open the drain port of the colostomy Pouch.

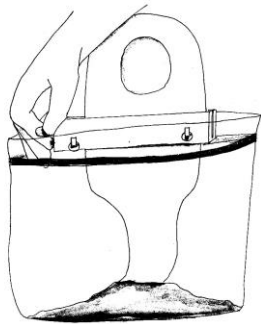

3. Now attach the front of the SOS Collection Bag to the front of the SOS. While twisting the knob apply gentle downward pressure on the SOS “tractoring” it down the colostomy pouch, emptying the contents of the pouch into the SOS Collection Bag.



4. Detach the SOS Collection Bag, seal it and dispose of it.



5. Clean any residue from the SOS using warm water and soap.