



Open The Traveler Kit, Remove Contents, Put on Gloves



Unlock The Traveler and slide-up to where rollers are just below fill port and snap closed with knob toward center of torso

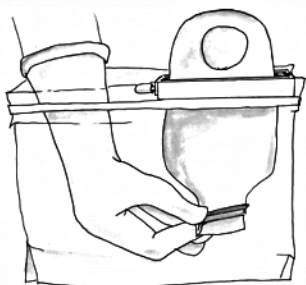

Reach into Collection bag and open drain port

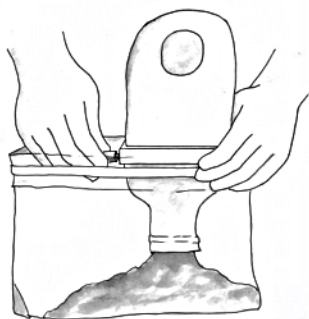

While twisting knob & applying slight downward pressure "tractor" rollers down bag pushing contents into Collection bag.



Fold up sides of Collection bag and place it aside. Use wipes to clean drain port. Close drain port.

Place wipes and gloves in Collection bag. Seal the top and dispose of the bag.

www.sanitaryostomysystem.com
7450 Vernalis Road, Atascadero, CA 93422